

Display Monitor Image Quality Evaluation

PERFORMED MONTHLY

Facility: _____ Room # _____ Unit: _____

Month Day (Year) Technologist's Initials	1	2	3	4	5	6	7	8	9	10	11	12
SMPTE Pattern Check:												
Is each gray level step of the pattern distinguishable from the adjacent step?												
0%-5% low contrast target discernible?												
95%-100% low contrast target discernible?												
Are the alphanumeric characters sharp and in focus?												
Are the high-contrast line-pair images in each of the corners and center distinguishable and sharp?												
Analysis of Glass and Screen Defects:												
No visible scratches?												
No visible black points?												
No visible non-uniformities?												
Other?												


Pass = v ; Fail = F;

Does not apply = NA

Window Width: 100

Window Level 1024

Otherwise, Manufacturer's recommended values should be used.

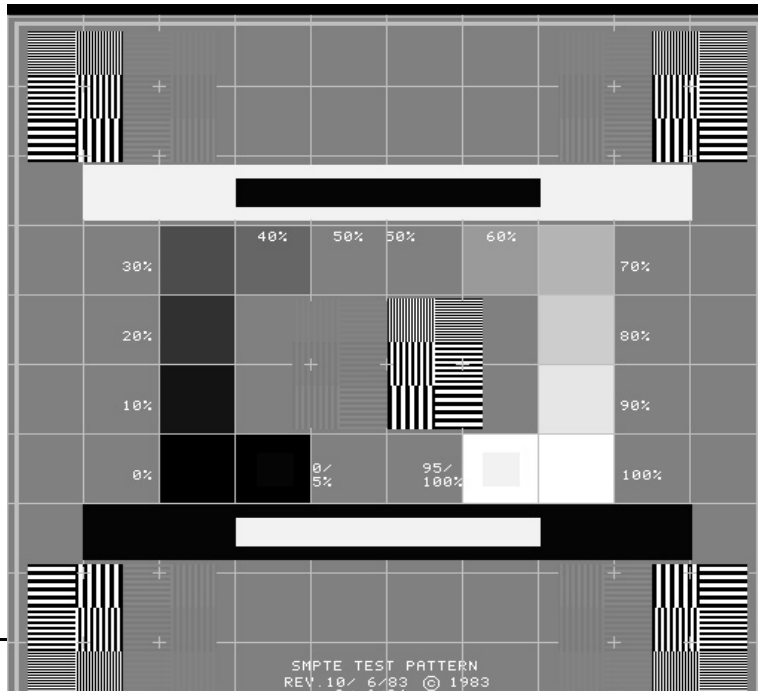

Reviewed by:

Qualified Medical Physicist

Date of Review

Provided by: Radiation Services, Inc.